

LiquiBand® Optima

The fast, precise and controlled topical skin adhesive.



**BETTER WOUND CLOSURE
STARTS WITH A BETTER TOOL**

LiquiBand® Optima Topical Skin Adhesive has a unique winged applicator with precision tip for accuracy and control, delivering excellent cosmetic results.

✓ Precision Applicator

The precision tip delivers accurate application, making the device ideal for traditionally difficult areas. The in-built flow control wings allow the clinician to easily control the volume of adhesive applied.

✓ Quick 10 Second Dry Time

Rapid wound closure in approximately 10 seconds offering faster patient turnaround. Only one application required for strong and secure closure.

✓ Ambient Storage

Can be stored between 5°C and 25°C reducing the need for refrigeration.

✓ Microbial Barrier Protection

Adhesive acts as a protective waterproof barrier, allowing patients to shower without compromising the wound closure. The waterproof, microbial barrier reduces the risk of infection.

✓ No-Sting Adhesive


Provides comfortable closure, minimising trauma — ideal for children.

✓ Outstanding Patient Benefits

Excellent cosmetic results combined with minimal discomfort throughout treatment and healing.

✓ Pre-Assembled Unique Winged Design

Ready-to-use straight out of the box. To activate the device, simply squeeze the wings until a cracking sound is heard. The transparent reservoir provides visualisation of remaining adhesive throughout the application process.

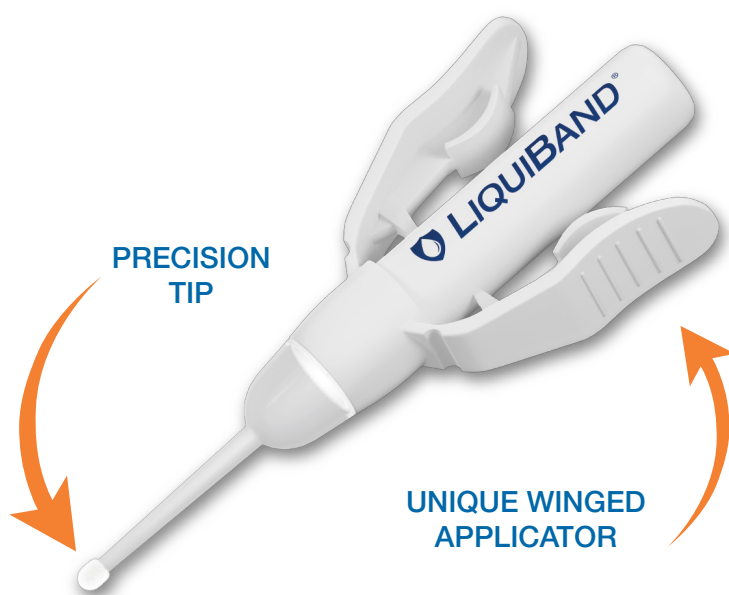

Fast, precise and controlled skin closure
LiquiBand® Optima is ideal for use on a
range of acute wounds including scalp
and facial lacerations.



Looking after your team

We are a proud independently owned Australian company

BETTER WOUND CLOSURE STARTS WITH A BETTER TOOL



LiquiBand® Optima Application Guide

Wound closure in
3 easy steps



STEP ONE

- ▶ Ensure that wound is thoroughly clean and dry prior to closure
- ▶ While holding upright, squeeze the applicator wings until a cracking sound is heard



STEP TWO

- ▶ Invert the tip and gently squeeze wings to prime the device
- ▶ When the tip fills with adhesive, the device is ready to use



STEP THREE

- ▶ While holding the wound edges together, apply an even coat of adhesive over the full length of the wound
- ▶ Continue to hold the wound edges together for 10 seconds

** If being used in areas of dense natural hair, take care when applying as hair follicle damage may occur if over applied.*

Experience the LiquiBand Difference


LiquiBand Optima
mini .35g Applicator

ADMOPT002

Box/10

Compare and save.

Contact us for pricing, information and special offers.

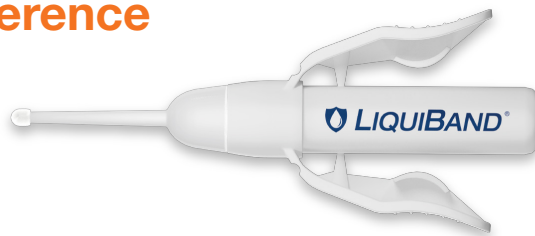

Try the topical skin adhesive with the best features



LiquiBand® Optima

Available exclusively from
Team Medical Supplies
and BP Medical.



Order from Team Medical Supplies

Phone 1300 22 44 50

Fax 1300 22 44 60

www.teammed.com.au

info@teammed.com.au



Order from BP Medical (WA)

Phone 1300 33 55 60

Fax 1300 33 55 70

www.bpmedical.com.au

info@bpmedical.com.au

