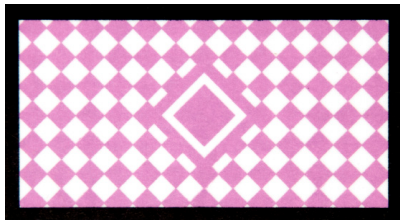
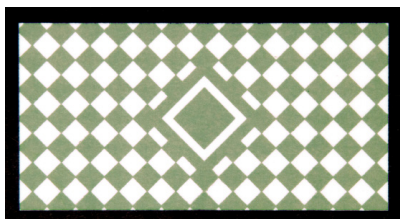
INTERPRETATION IMAGES

1

2

3

This is one example of a fail

ENGLISH

Indications for Use

The Geringe Assured Bowie-Dick Test Card is used for daily monitoring of pre-vacuum steam sterilizers operating at 134°C. The stated values 134°C for 3.5 minutes are valid for laboratory test cycles as specified and required by the indicator standard EN ISO 11140-4. A true hospital Bowie-Dick test cycle, as specified by EN 285, is different from the EN ISO 11140-4 test cycle and is required to mirror the routine hospital sterilizer cycle. To override differences between indicator test cycle and hospital Bowie-Dick test cycles, EN 285 allows variation of the plateau period. Consequently, Geringe Bowie-Dick cycles are designed with a plateau period less than 3.5 minutes for equivalency with both standards. This Geringe Assured product is tested and verified to have equivalent and approved performance at both EN ISO 11140-4 subatmospheric and trans-atmospheric test cycles and Geringe Bowie-Dick cycles as specified by EN 285.

Instructions for Use

1. Run a warm up cycle to properly heat the sterilizer and the holder.
2. Place the test card into the reusable holder with the indicator ink pattern facing up.
3. Place the test card and holder on the bottom rack of the sterilizer approximately 10 cm above the drain.
4. Run a Bowie-Dick test cycle as specified by the sterilizer manufacturer. Note: A short dry time of 2 to 5 minutes will not affect the test.

Interpretation

1. Remove the holder from the sterilizer as soon as the cycle has finished. Caution: the holder may still be hot and should be handled with care.
2. Remove the indicator card from the holder.
3. If the sterilizer is functioning properly, the indicator graphic will show a color change from purple to a uniform green throughout the entire indicator image.
4. When excessive residual air or non-condensable gases are present (major failure), the indicator will not be a uniform green color but remain purple in the center or a shade of gray. This may require a sterilizer service call.
5. A brown or non-uniform green color may indicate problems with steam conditions in the sterilizer chamber other than non-condensable gases or residual air.

Color Change

- 1 Unprocessed
- 2 Pass
- 3 Fail (Any presence of purple indicates a fail)

Storage

Store at normal room temperature 10°–38°C and 10–60% Relative Humidity. Protect from moisture and excess humidity.

Expiry Date

The expiry date is printed on the product packaging. Do not use after the expiration date on the product packaging.

Interfering Substances or Conditions

There are no known interfering substances or conditions that could affect the intended use of the indicator or adversely affect the indicator performance.

Release of Toxic Substances

The indicator releases no known toxic substances in sufficient quantities to cause either a health hazard, or a hazard to the intended properties of the product being sterilized before, during or after the sterilization process.

Declaration of Conformity

The Geringe Assured Bowie-Dick Test Card conforms to ISO 11140-4 and EN ISO 11140-1 Type 2 Indicators for use in specific tests.

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